

# DRI OCT Triton series

Swept Source Optical Coherence Tomography



PERFORMANCE  
YOU CAN COUNT ON

 **TOPCON**

# See. Discover. Explore.

The diagnostic power of Swept Source OCT  
**Deep Range Imaging.**

“Swept source adds a new dimension to OCT. The TOPCON DRI Swept Source OCT is easy to use, provides unique clinical information, and has improved my practice. For the first time, we can in-vivo visualize not only the vitreo-retinal interface but also the cortical vitreous which is important at the time when more and more therapies are delivered via intra-vitreous injections. Deeper imaging brings choroidal thickness, helping guide my clinical decisions. Seeing more helps guide my therapy and allows me to treat more effectively. I find Swept Source OCT an essential tool to look for biomarkers of disease regression or progression.”

*Prof. P. E. Stanga,  
Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at NIHR/  
Wellcome Trust Manchester CRF & University of Manchester*

## Welcome to the New Frontier in OCT Imaging

The DRI OCT Triton combines the world's first swept source OCT technology with multimodal fundus imaging. Multimodal All-in-One fundus imaging tool will bring the next level of diagnostic capability to you and your patients.

### Unprecedented image quality

Triton's swept source with its fastest scanning speed and longer 1,050nm wavelength results in stunningly clear, detailed images, even into the deepest layers of the eye with short acquisition time. You will not only see the retina and vitreous, but also the choroid and the sclera like never before.

### Remarkable diagnostic capability

Seeing deeper makes it possible to have a better understanding of many ocular pathologies, and may provide the advantage of early disease detection and monitoring. Combined with unique features such as OCT Angiography and En-face imaging, Triton empowers you to take proactive steps to preserve your patients' eye health.


### Greater clinical efficiency

A wealth of automated and intuitive functions, including single-scan captures and the new SMARTTrack™ system, are designed to optimize your practice workflow by simplifying data capture, analysis, and follow-up.




See Deeper. See More.

Proliferative diabetic retinopathy

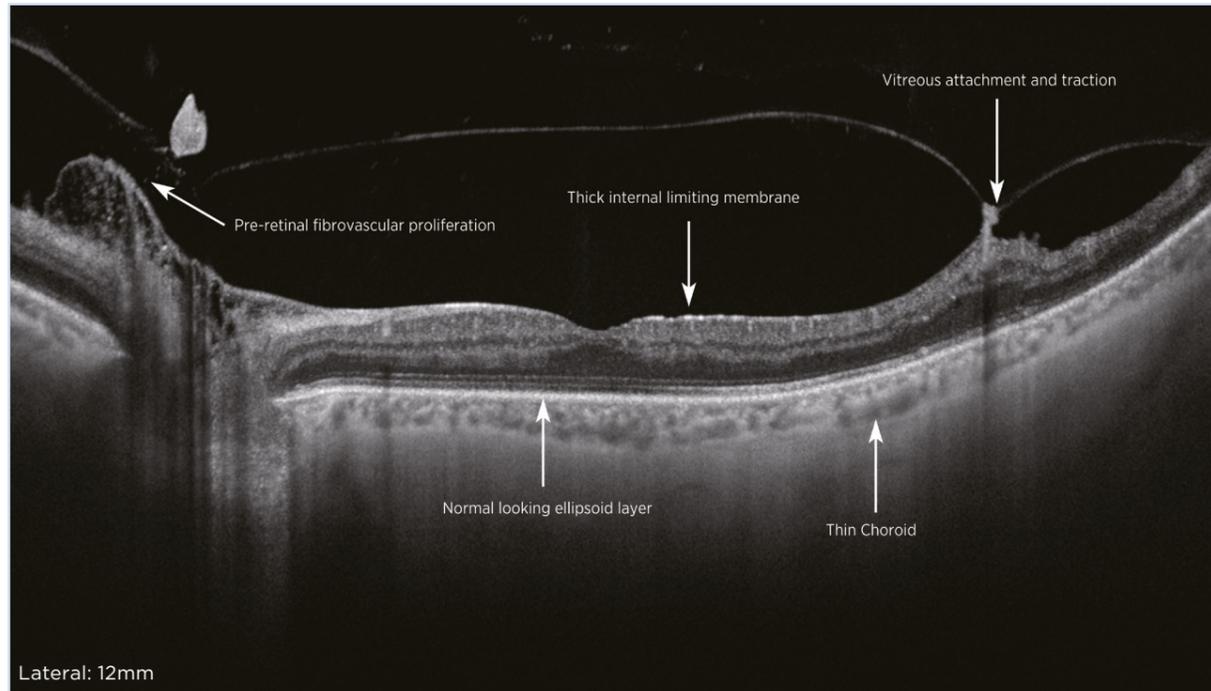

Courtesy: Prof. P. E. Stanga, Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at N IHR/ Wellcome Trust Manchester CRF & University of Manchester

Central serous retinopathy

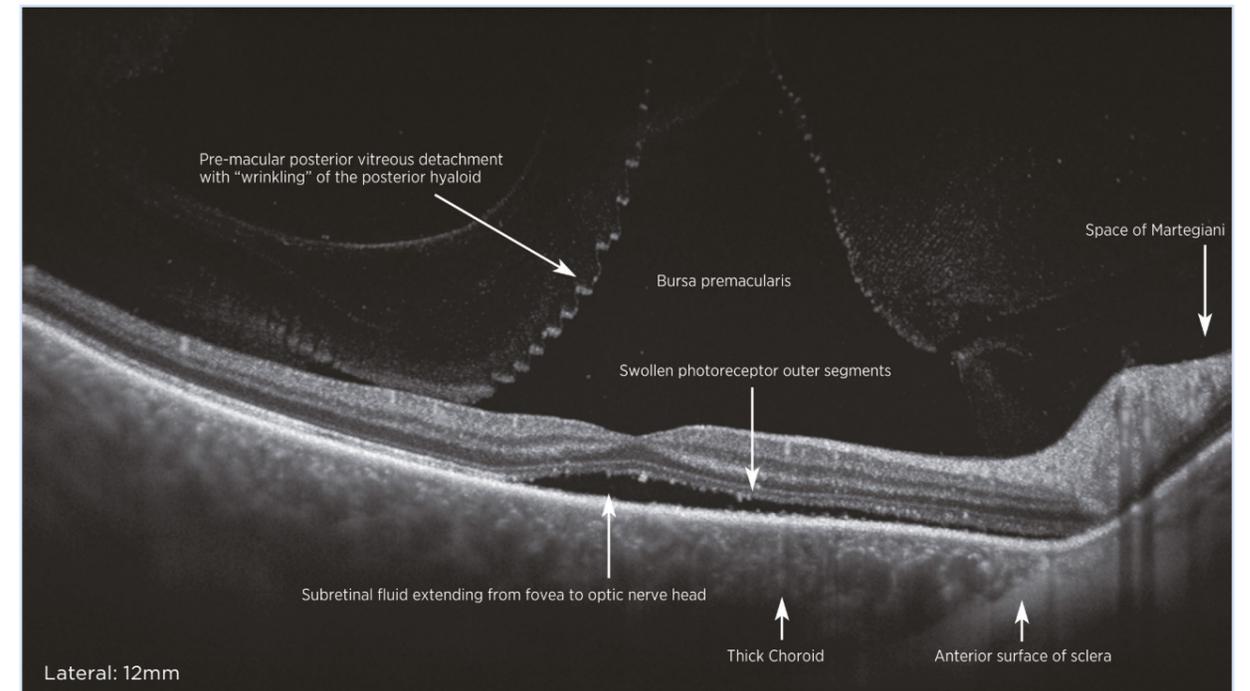


Courtesy: Prof. P. E. Stanga, Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at N IHR/ Wellcome Trust Manchester CRF & University of Manchester



Courtesy: Prof. P. E. Stanga, Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at N IHR/ Wellcome Trust Manchester CRF & University of Manchester

\* FA photography and FAF photography can be performed using only DRI OCT Triton plus.

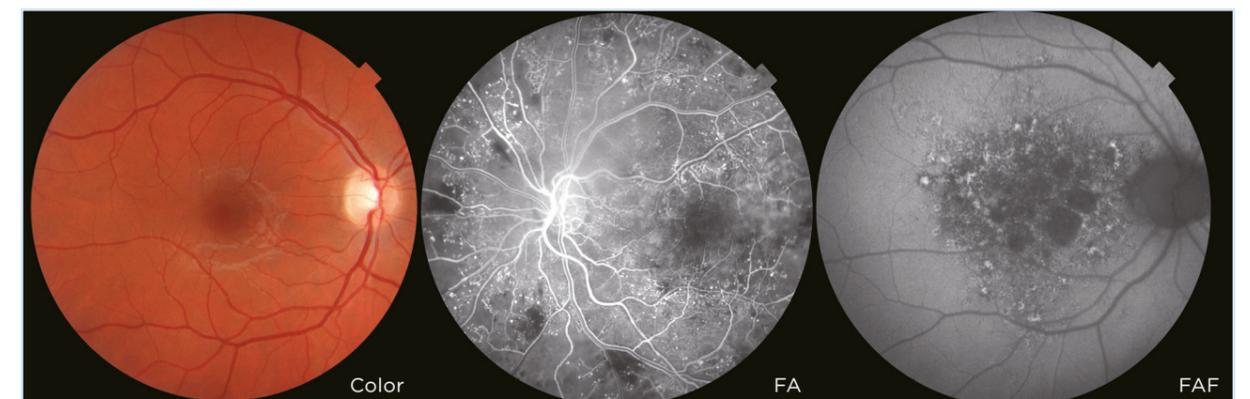


Courtesy: Prof. P. E. Stanga, Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at N IHR/ Wellcome Trust Manchester CRF & University of Manchester

\* FA photography and FAF photography can be performed using only DRI OCT Triton plus.


See Deeper. See More.

### Pathological myopia

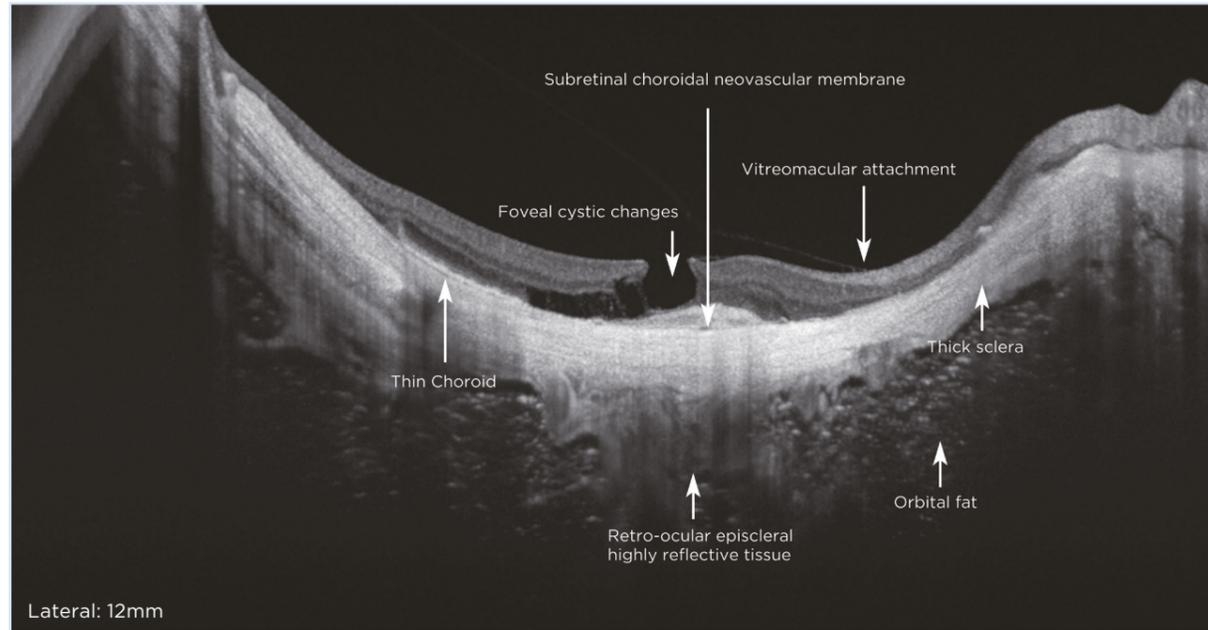


Courtesy: Prof. P. E. Stanga, Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at N IHR/ Wellcome Trust Manchester CRF & University of Manchester

### Macular pucker


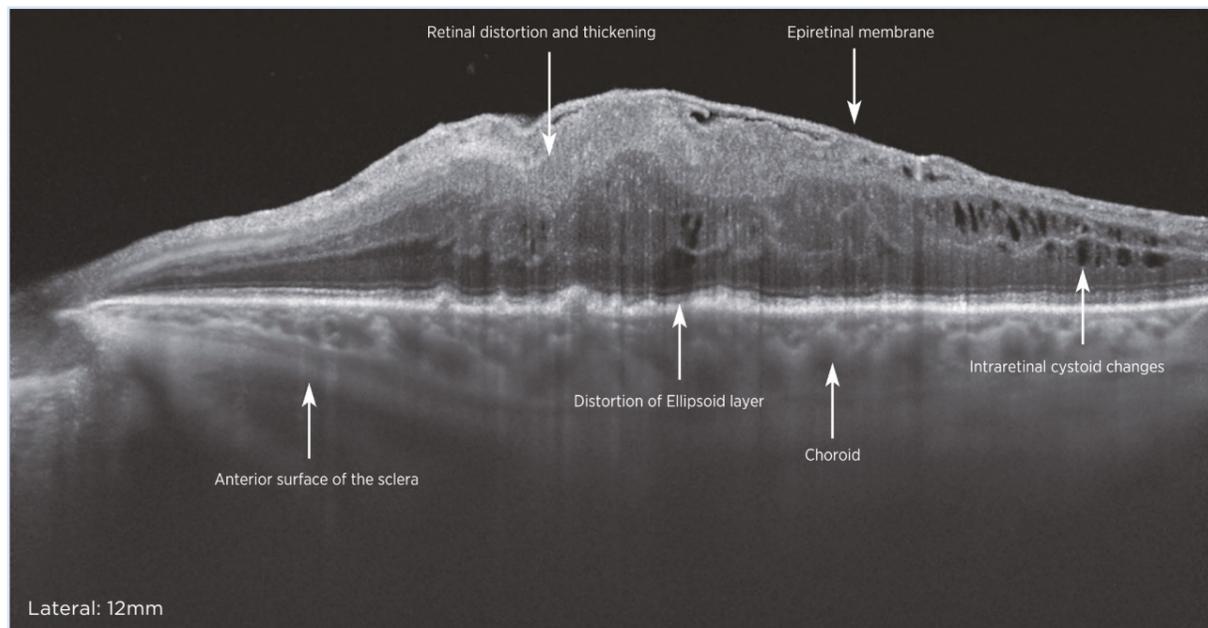


Courtesy: Prof. P. E. Stanga, Manchester Royal Eye Hospital, Manchester Vision Regeneration (MVR) Lab at N IHR/ Wellcome Trust Manchester CRF & University of Manchester

### Image through cataract

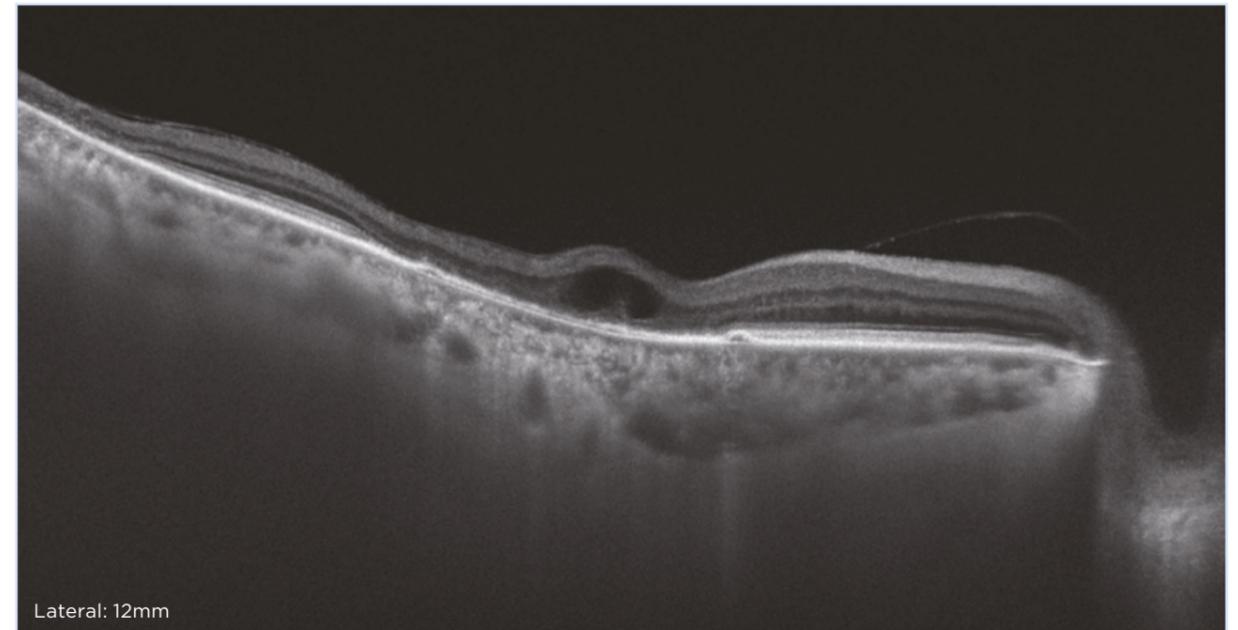

a)

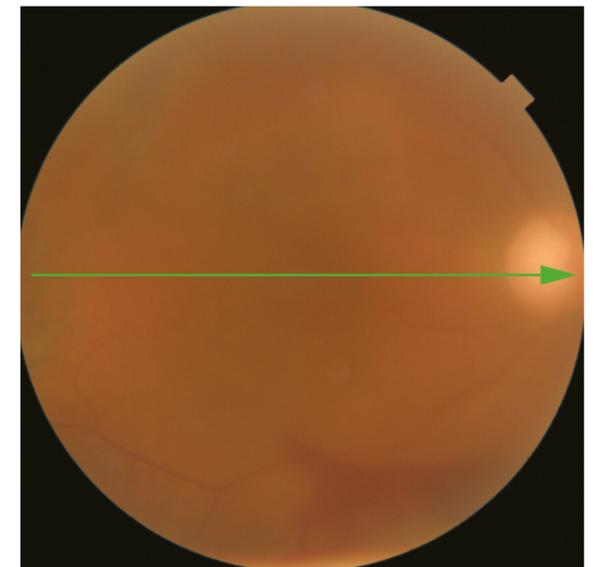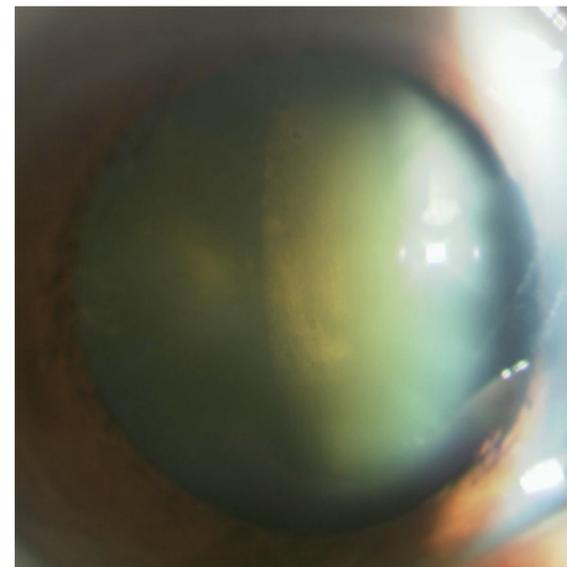

b)

c)

a),b),c) Courtesy: Kazuya Yamagishi, MD (Hirakata Yamagishi Eye Clinic, Japan)


# Swept Source Takes OCT Technology to a Whole New Dimension.

## Envision the possibilities

DRI OCT Triton's swept source OCT technology and long wavelength 1,050nm light enable both a deeper imaging range and a better tissue penetration, compared with the conventional spectral domain OCT. The OCT images captured by DRI OCT Triton are clearly described from vitreous, retina and choroid in a single capture, without degrading OCT image quality in deeper depth. The longer wavelength reduces risks of light attenuation by cataract and vitreous opacity, making OCT imaging more feasible for the patients with those diseases. Advantages of DRI OCT Triton's technology improvement over the conventional spectral domain OCT will provide more information for your diagnosis and more comfort for your patients. It's advanced technology that everyone can appreciate.

## Optimized wavelength for retinal imaging: 1,050nm

The longer wavelength light provides better tissue penetration, allowing visualization into the deepest layers of the eyes – even through cataracts, hemorrhages, and blood vessels.

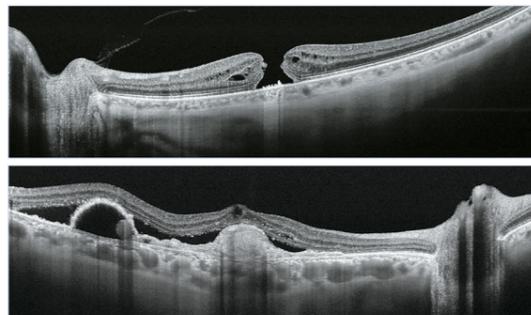


Courtesy: Professor Jose Maria Ruiz Moreno, University of Albacete, Spain.

## Swept Source OCT technology; the world's fastest scanning speed


Swept source technology provides the world's fastest 100,000 A-scan/ sec, in the current conventional Spectral Domain OCT. The faster scanning speed enables capturing a clear B-scan by acquiring more A-scans within a given image acquisition time. It helps to reduce error of the involuntary eye movement.

\*According to the TOPCON survey May 2015

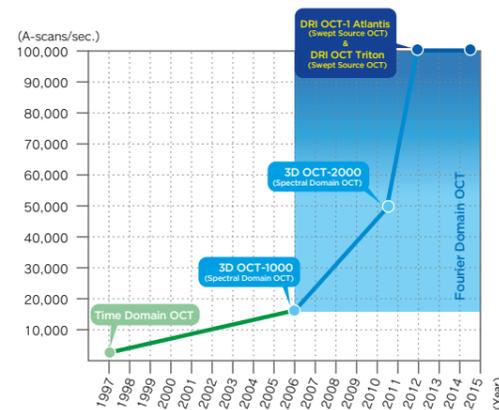

## Invisible scan lines


The invisible 1,050nm wavelength light helps patient to concentrate on the fixation target during the measurement, reducing involuntary eye movement. It supports more efficient workflow in a practice by reducing re-scan.

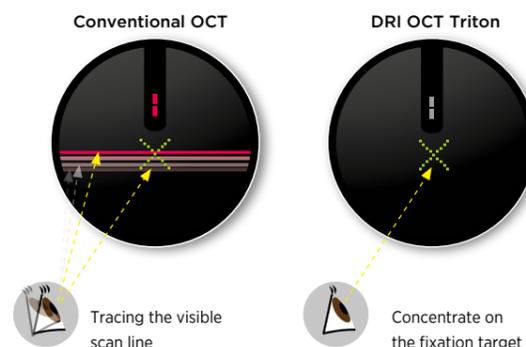


## Swept Source OCT Angiography

OCT Angiography<sup>1,2,3</sup> is a novel and non-invasive imaging technique to visualize the microvascular network. It is now available any time you need it. The optional OCT Angiography module offers non-invasive observation of the microvascular structures reducing the need for conventional fluorescein angiography.

- » By utilizing cutting-edge swept source technology with a 1,050nm wavelength, high-quality OCT Angiography images are acquired
- » Easier recognition of abnormalities by using layer by layer “tissue peeling” intuitive graphical user interface
- » Improved patient comfort<sup>4</sup> - no dyes or dilation required, rapid capture with our intuitive graphical user interface
- » Direct comparison and registration with fundus images in IMAGENet 6<sup>5</sup>

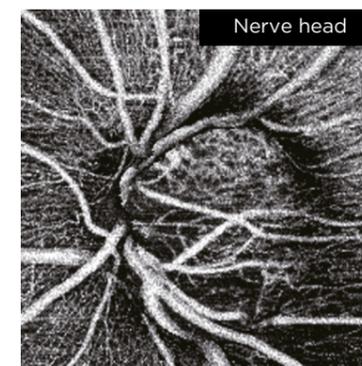


Nerve head  
Glaucoma

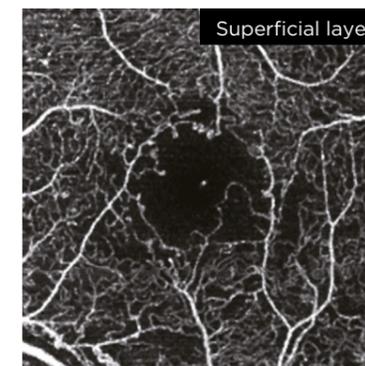


Superficial layer  
Diabetic Retinopathy

Courtesy: Dr. A. Ishibazawa and Prof. A. Yoshida (Asahikawa Medical Univ., Japan)

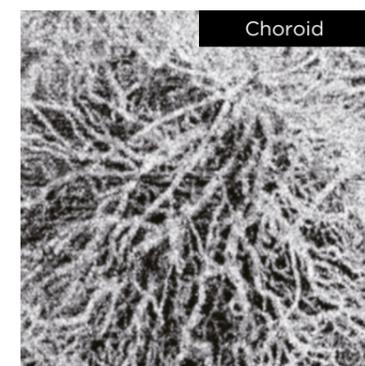


Choroid  
CNV

Courtesy: Srinivas R. Sadda, M.D., Laura Kuehlewein, M.D., Doheny Eye Institute, UCLA

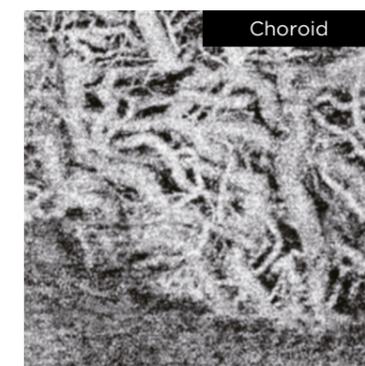

Choroid  
GA

Courtesy: Srinivas R. Sadda, M.D., Laura Kuehlewein, M.D., Doheny Eye Institute, UCLA

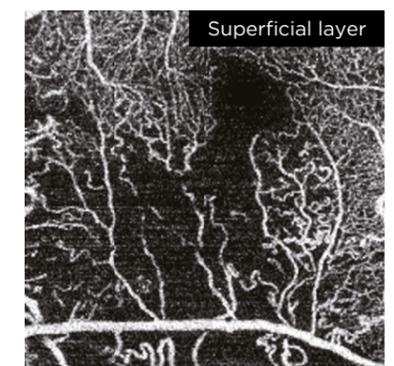


Superficial layer  
BRVO

<sup>1</sup> OCT Angiography scanning line may be visible during capture to some people under certain conditions  
<sup>2</sup> Viewing an OCT Angiography image is possible only in combination with IMAGENet 6  
<sup>3</sup> OCT Angiography is optional software  
<sup>4</sup> Compared to conventional fluorescein angiography  
<sup>5</sup> Optional software


# Improved Clinical Efficacy With Sophisticated Analysis Functions.

## en face OCT imaging


en face imaging allows for independent dissection of the vitreoretinal interface, retina, retinal pigment epithelium (RPE), and choroid by flattening B-scan image. Pathology throughout the posterior pole can be studied and correlated with a patient's symptoms, their abnormality, and its progression.

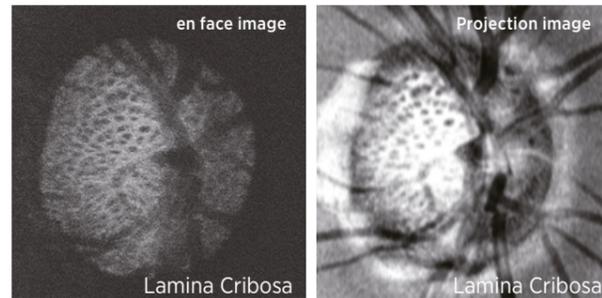

Courtesy: Prof. T. Nakazawa, Tohoku University, Japan

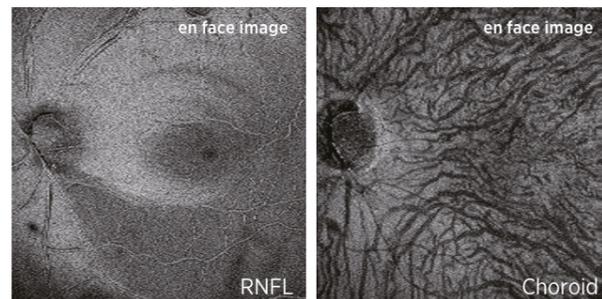

Courtesy: Prof. T. Nakazawa, Tohoku University, Japan

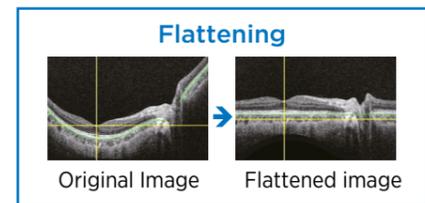


## To visualize vitreous

### Dynamic Focus™

To enhance weak signal in vitreous part, DRI OCT Triton's advanced capturing technique, named "Dynamic focus", enables the acquisition of high-quality and uniform image quality with a focus uniformly focused across the entire imaging range.

Clear image in all area

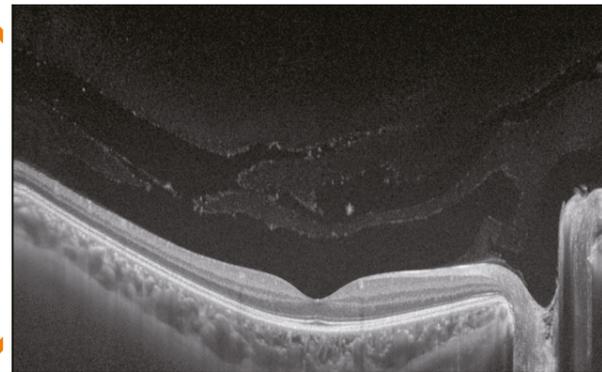


### EVV (Enhanced Vitreous Visualization™)

Improved vitreous visualization with DRI OCT Triton helps assess the nature of vitreoretinal interface abnormalities. Contrast can be quickly adjusted to the needs of the physician, depending on the area of greatest interest.




## Normative database with swept source OCT

DRI OCT Triton includes a normative database for statistical comparison of the thickness maps and parameters. By comparing individual measurement value with the corresponding normative range, the DRI OCT Triton provides you with a powerful reference tool to enhance your analysis in both research and patient diagnosis.

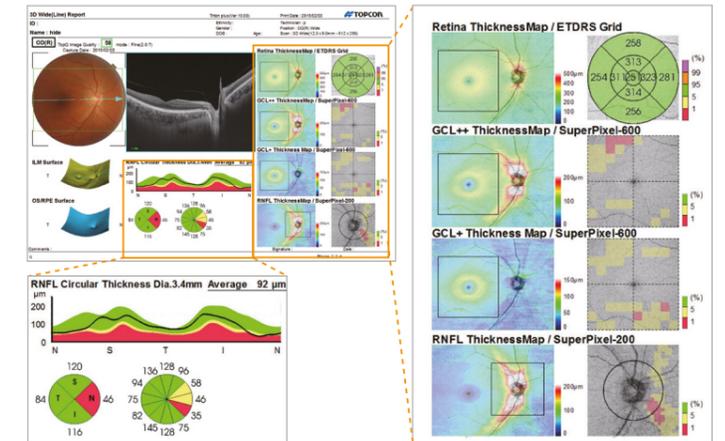

## 7 boundaries segmentation/5 layers thickness map/caliper function

Retinal tissue layers are automatically segmented by the Topcon Advanced Boundary Software (TABSTM), enabling to quantify the internal thickness for change analysis.

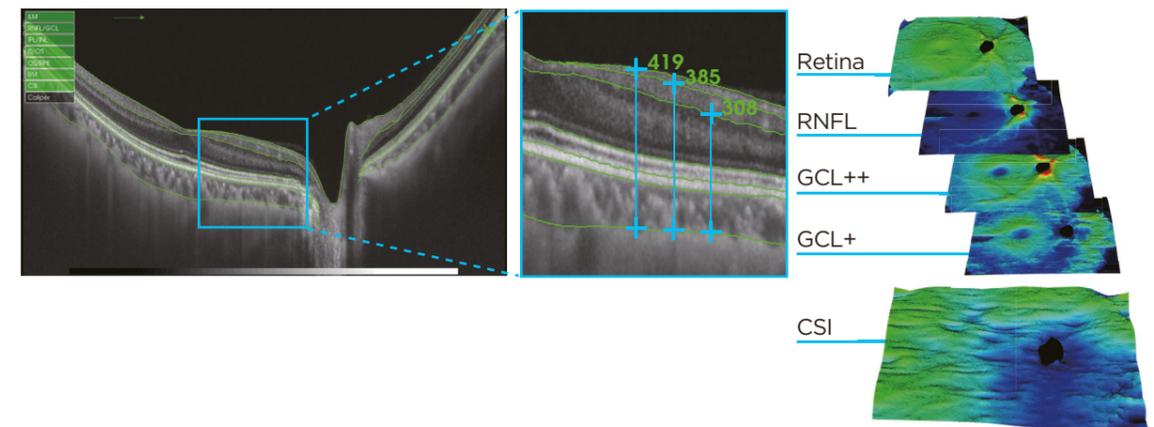

## Accurate choroidal thickness maps


The choroid reveals valuable information about the health of the eye. High-speed choroidal thickness maps are crucial for early disease recognition and monitoring of inflammatory abnormalities. For example, a thin choroid can be an indication of myopic or choroidal atrophy. A thick choroid may indicate the presence of choroiditis, Central Serous Chorioretinopathy (CSCR) or hyperopia.

- Retina** between the ILM-OS/RPE boundaries
- RNFL** between the ILM-RNFL/GCL boundaries
- GCL+** between the RNFL/GCL-IPL/INL boundaries
- GCL++** between the ILM-IPL/INL boundaries
- CSI** between the BM-CSI boundaries or ILM-CSI boundaries


# DRI Meets Multimodal Fundus Imaging: See the Whole Picture.

## Swept Source OCT incorporates multimodal fundus imaging

DRI OCT Triton can acquire the OCT and fundus image in a single capture. Pinpoint Registration™ identifies the location of B-scan on the fundus image. Clear comparison between the B-scan and fundus image can support clinical efficiency during diagnosis.

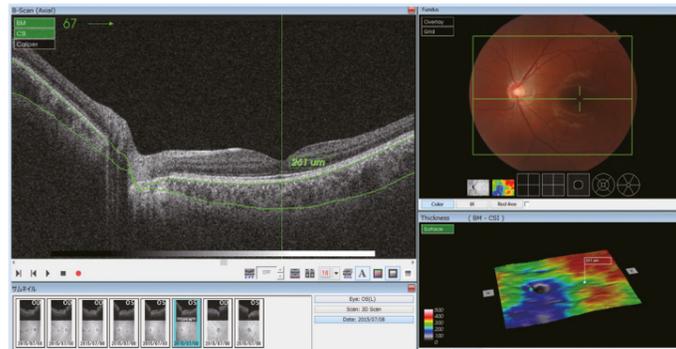

OCT + Color fundus

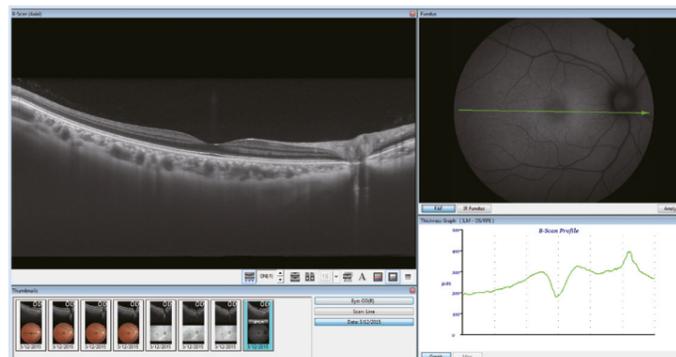


OCT + FAF

## High-quality fundus images


The DRI OCT Triton offers a color, non-mydratric fundus image. Fundus Angiography (FA) and Fundus Autofluorescence (FAF) are available to meet your needs. The all in one device supports efficient workflow in practice.\*

\*DRI OCT Triton plus :  
OCT /Anterior OCT (Option)/ OCT Angiography (Option)/Color / Red-Free / FA / FAF  
DRI OCT Triton :  
OCT /Anterior OCT (Option)/ OCT Angiography (Option) / Color / Red-Free

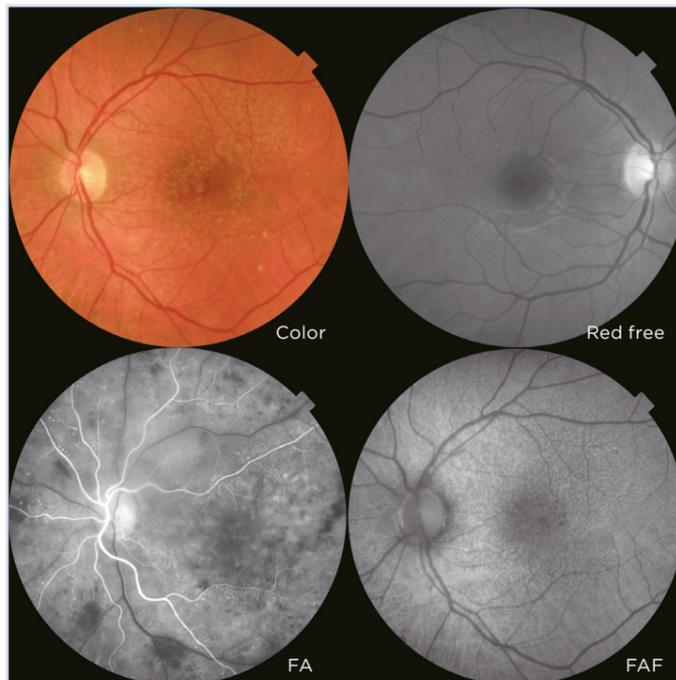


## Stereo photography

Three dimensional visualization of color fundus image can be achieved by acquiring the images in stereo photography mode. Triton's monitor guidance provides quick and easy operation with auto alignment function for a stereo pair.

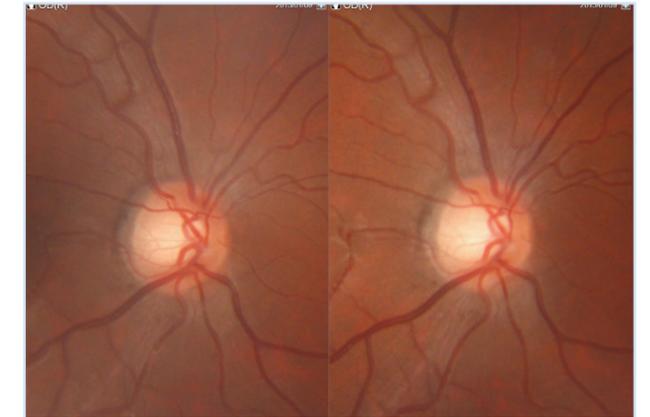

## Panoramic wide field photography

In addition to macula and disc image, DRI OCT Triton allows to acquire wide coverage of the retina. With these images, a panoramic graphic can be created on the optional software.

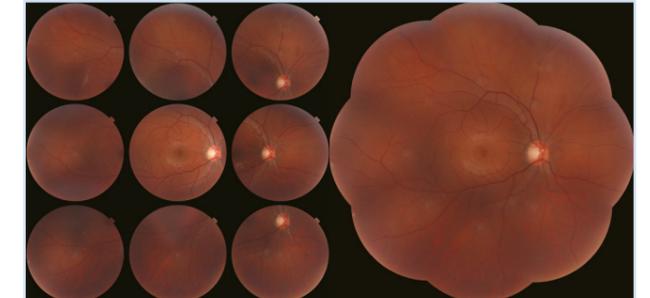


# Smarter Tracking. Smarter Workflow.

## SMARTTrack™ makes tracking ingeniously simple


The new SMARTTrack™ tool enhances the tracking and follow-up ability of Triton with a variety of functions designed to enhance its user-friendliness:

- » Fundus-Guided Acquisition (FGA)
- » Follow-up Function
- » Tracking photography

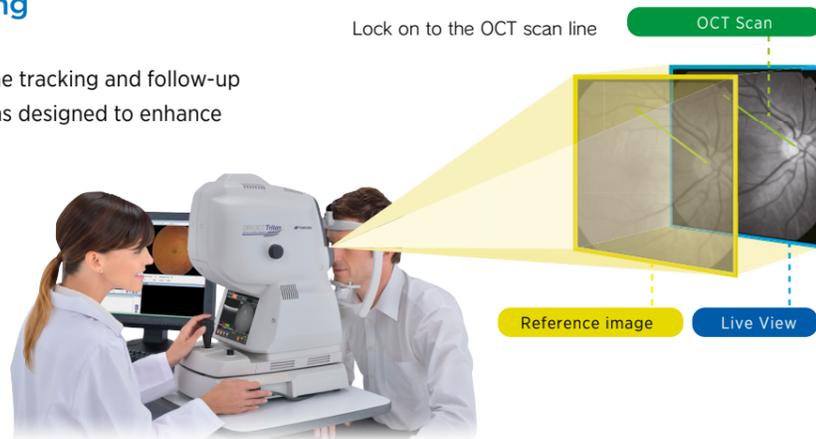


## Fundus Guided Acquisition (FGA)

OCT scan location can be easily set by selecting the scan area on the fundus image, making fundus abnormalities viewable with no additional operator steps required. With FGA, the operator can choose to take or import a fundus image, select the scan location, and automatically acquire a B-scan.

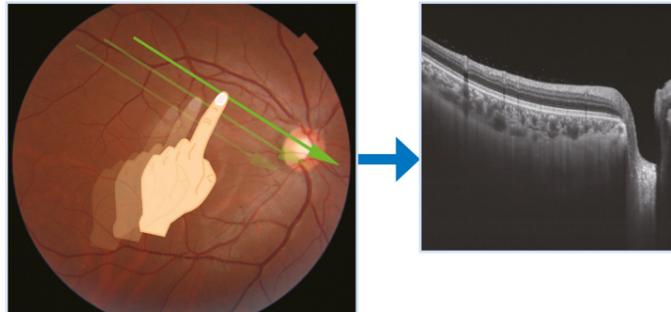

## Follow-up function

This function allows you to retrieve and reanalyze the same location at follow-up, for comparison of past and current images. All an operator needs to do is simply select the past data, and Triton automatically captures the same area. Comparison of the same area supports diagnostic accuracy.

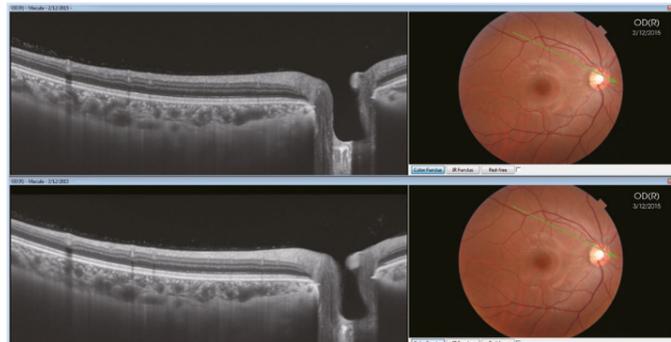

## Motion Correction / Compensation / Rescanning Function

### Motion Correction


Corrects the Z direction movement

### Compensation Function

Tracks the eye and then compensates for the X direction movement.

### Rescanning Function

The scanning area may be missed due to Y direction eye movement. In such a case, the rescanning function automatically activates. It automatically rescans the missing scan area.

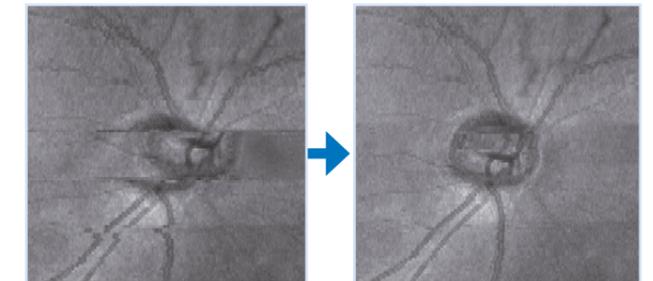


Before compensation

After compensation

## Alignment navigation

When an operator wishes to acquire an image, Triton's monitor guides the operator to reduce potential errors and make operation simple

- » Auto focus and auto shoot, in color/FAF mode
- » Auto focus, auto-Z and Z-lock function, in OCT mode

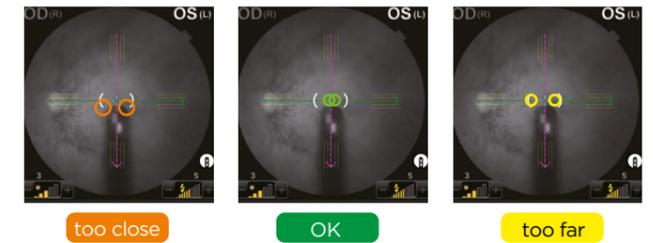

too close

OK

too far


## The small pupil solution

### Live Fundus View

The fast scanning speed allows the Triton to create a live en face fundus image, an ideal tool for precisely locating the scan position. Therefore the disc, retinal vessels and scanning position are easy to see, even in patients with small pupils.

### OCT capture mode without retinal photography


Triton can also capture a 3D scan, with or without color fundus photography, to avoid a miotic response and better meet the needs of patients with small pupils.

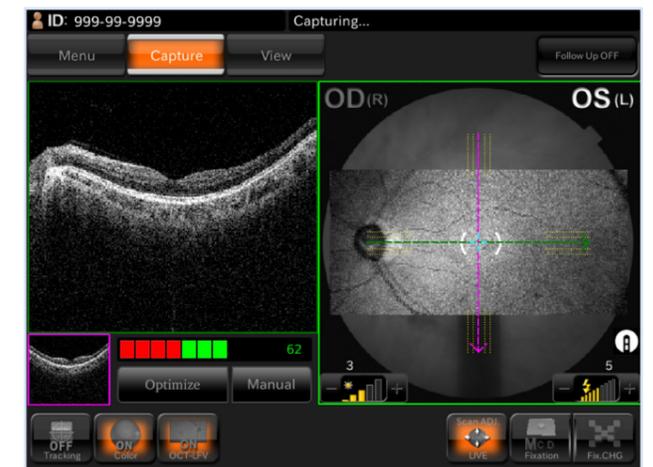

# Comprehensive Data Analysis at Your Fingertips.


## Powerful reporting for enhanced decision making

Triton's comprehensive data analysis options make it easy to monitor patients with individual measurement data and corresponding normative data range. Therefore, you can have better support for the diagnosis, treatment and management of patients with glaucoma and macular degeneration, as well as other conditions.

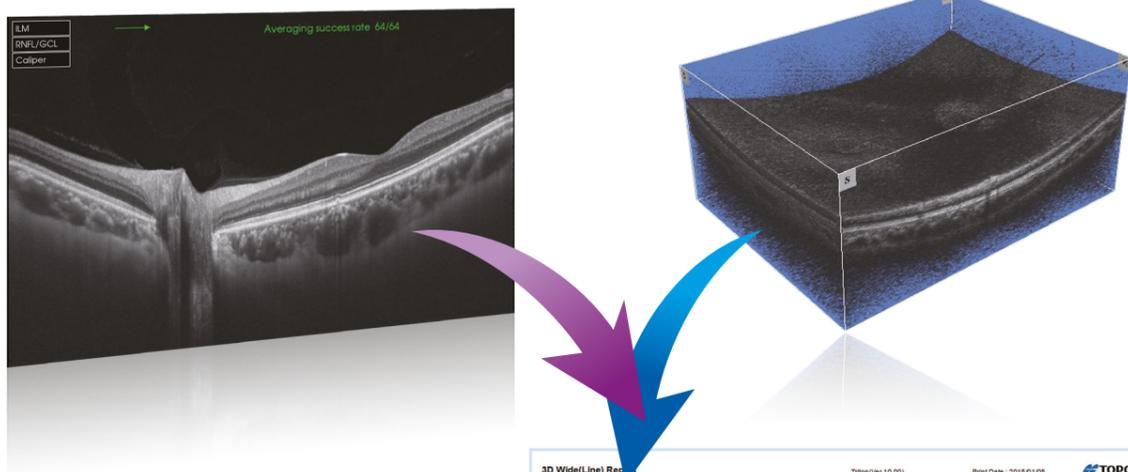


## Combination scan

This new scan pattern provides both 3D wide scan (12mm x 9mm) and Line / 5 line cross / radial scan. Now TOPCON OCT models offer the option to capture B-scan and 3D images at the same time. The new combination scan provides a thickness map, 3D image and an overlapped clear B-scan image in a single capture.

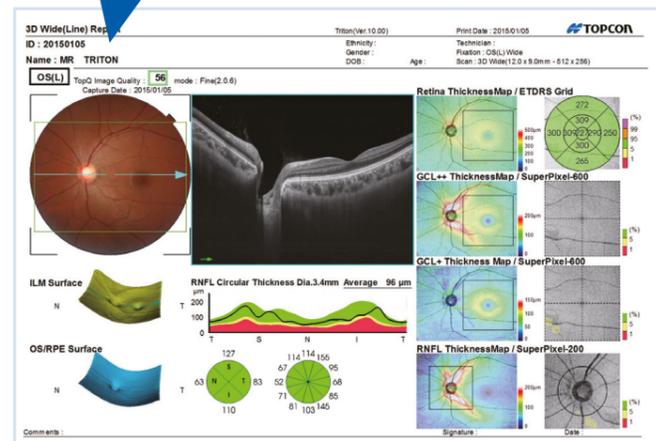


## Comprehensive Analyzed Data

Easy-to-read & easy-to-understand report templates.

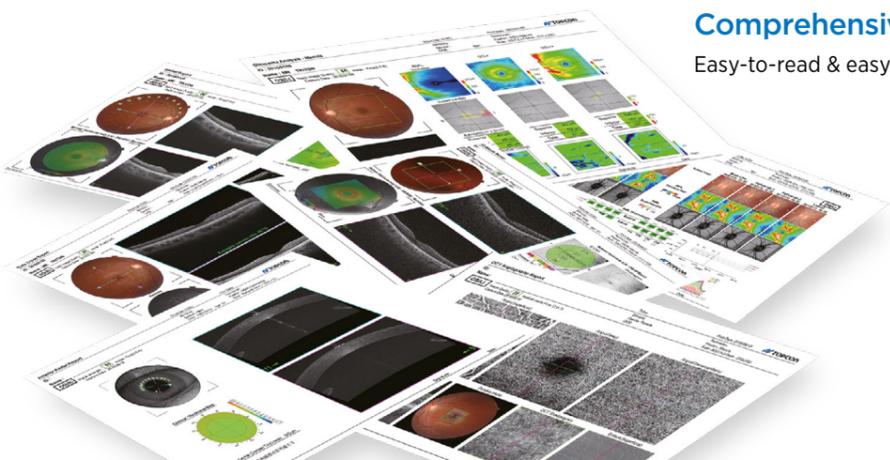

## Anterior segment imaging

Triton has optional anterior imaging capabilities to enhance anterior segment data collection. The anterior segment attachment ensures sharp images, even in the periphery and the anterior chamber.

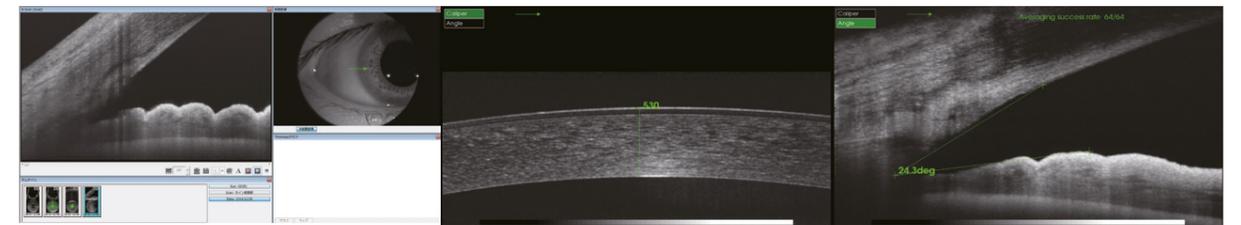

## Anterior segment attachment kit\*

- 1 Anterior segment attachment
- 2 Head rest attachment




\*Observation & photography of the anterior segment can be performed only when the optional anterior segment attachment kit is used.

## Image samples


### OCT image B-scan length 16mm




### Anterior segment in Radial scan

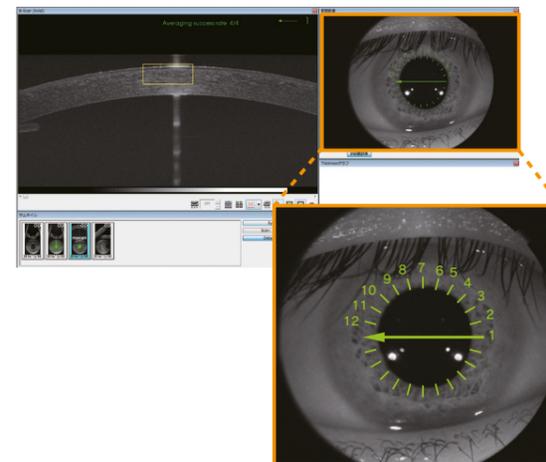

### Anterior segment in 3D scan

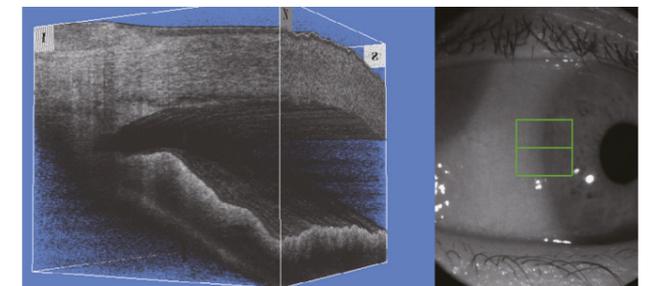

# View Any Data<sup>1</sup>. Anywhere<sup>2</sup>. Any Time.

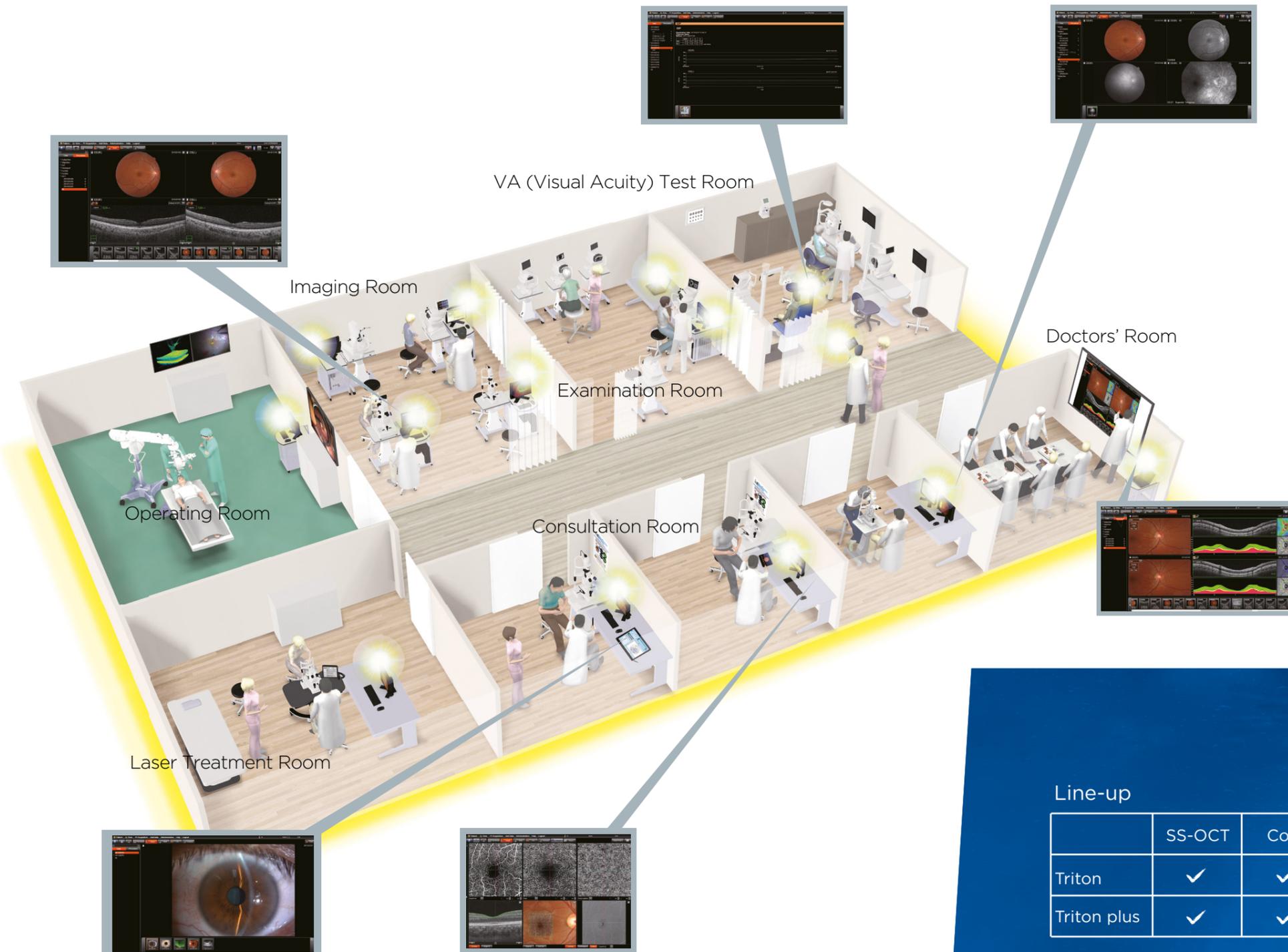

## Transform the way you manage ophthalmic data and images

### Widely Connected

IMAGEnet 6 uses a web-based application, your patient data can be accessed from any PC or tablet in your practice or hospital network.

With accessibility from any devices which you pick up at that time, more convenience and more flexibility will support your efficient work flow

### Impressively Comprehensive

Now you can review all data captured by any TOPCON device with one software application.\*<sup>2</sup> All the data you need can be shown on one screen to support a deeper understanding of your patient's condition

### Remarkably Easy

The data you need is just a click away. IMAGEnet 6 was developed to give you a simple and efficient way to review data with informative one-page Graphical User Interface(GUI) and fast response time.\*<sup>3</sup> Web-based application requires no installation to each device for easy maintenance. It allows you to spend more time on what matters-your patients.

<sup>1</sup> TOPCON instrument only.

<sup>2</sup> Internal hospital only

<sup>3</sup> Capture software is required.

<sup>4</sup> Compared to current OCT software.

## Line-up

	SS-OCT	Color	Red-free	FA	FAF	Anterior OCT	OCT Angiography
Triton	✓	✓	✓	—	—	Option	Option
Triton plus	✓	✓	✓	✓	✓	Option	Option

**DRI OCT Triton**  
3D OPTICAL COHERENCE TOMOGRAPHY

# Specifications

## Observation & Photography of Fundus Image

Photography Type	Color, FA*, FAF*, Red-free**
Picture Angle	45° Equivalent 30° (Digital Zoom)
Operating Distance	34.8mm
Photographable Diameter of Pupil	Normal:φ4.0mm or more Small pupil diameter: φ3.3mm or more

## Observation & Photography of Fundus Tomogram

Scanning Range (on fundus)	Horizontal Within 3 to 12mm Vertical Within 3 to 12mm
Scan Pattern	3D scan Linear scan (Line-scan/Cross-scan/Radial-scan)
Scan Speed	100,000 A-Scans per second
Lateral Resolution	20μm
In-depth Resolution	Digital: 2.6μm Optical function: 8μm
Photographable Diameter of Pupil	φ2.5mm or more

## Observation & Photography of Fundus Image / Fundus Tomogram

Fixation target	Internal fixation target : Dot matrix type organic EL The display position can be changed and adjusted. The displaying method can be changed. Peripheral fixation target : This is displayed according to the internal fixation target displayed position. External fixation target
-----------------	---

## Observation & photography of anterior segment\*\*\*

Photography type	IR
Operating distance	17mm

## Observation & photography of anterior segment tomogram\*\*\*

Operating distance	17mm
Scan range (on cornea)	Horizontal Within 3 to 16mm Vertical Within 3 to 16mm
Scan pattern	3D scan Linear scan (Line-scan/Radial-scan)
Scan speed	100,000 A-Scans per second
Fixation target	Internal fixation target External fixation target

## Electric Rating

Power Source	Voltage: 100-240V Frequency: 50-60Hz
Power input	250VA

## Dimensions / Weight

Dimensions	320-359 mm(W) X 523-554 mm(D) X 560-590 mm(H)
Weight	21.8kg (DRI OCT Triton) 23.8kg(DRI OCT Triton plus)

\* FA photography and FAF photography can be performed in only DRI OCT Triton plus.

\*\* In this digital red-free photography, the color image is processed and is displayed as a pseudo red-free photographed image.

\*\*\* Observation & photography of anterior segment can be performed only when the anterior segment attachment kit is used.



Not available for sale in the US.  
Not available in all countries, please check with your distributor for availability in your country.



Subject to change in design and/or specifications without advanced notice.

In order to obtain the best results with this instrument, please be sure to review all user instructions prior to operation.



TOPCON CORPORATION



TOPCON CORPORATION



TOPCON CORPORATION

## TOPCON CORPORATION

75-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8580, Japan. Phone:+81-(0)3-3558-2522/2502 Fax:+81-(0)3-3965-6898 www.topcon.co.jp

### TOPCON MEDICAL SYSTEMS, INC.

111 Bauer Drive, Oakland, NJ 07436, U.S.A.  
Phone:+1-450-430-7771 Fax:+1-450-430-6457 www.topconmedical.com

### TOPCON CANADA INC.

110 Provencher Avenue, Boisbriand, QC J7G 1N1 CANADA  
Phone:+39-02-9186671 Fax:+39-02-91081091 E-mail: info@topcon.it; www.topcon.ca

### TOPCON EUROPE MEDICAL B.V.

Essebaan 11; 2908 LJ Capelle a/d IJssel; P.O.Box145;  
2900 AC Capelle a/d IJssel; THE NETHERLANDS  
Phone:+31-(0)10-4585077 Fax:+31-(0)10-4585045  
E-mail: medical@topcon.nl; www.topcon.eu

### ITALY OFFICE

Viale dell'Industria 60; 20037 Paderno Dugnano; (Milano), ITALY  
Phone:+39-02-9186671 Fax:+39-02-91081091 E-mail: info@topcon.it; www.topcon.it

### DANMARK OFFICE

Praestemarksvej 25; 4000 Roskilde; DANMARK  
Phone:+45-46-327500 Fax:+45-46-327555  
E-mail: topcon@topcondanmark.dk www.topcondanmark.dk

### IRELAND OFFICE

Unit 276, Blanchardstown, Corporate Park 2 Ballycoolin Dublin 15, IRELAND  
Phone:+353-18975900 Fax:+353-18295915 E-mail: medical@topcon.ie; www.topcon.ie

### TOPCON S.A.R.L.

BAT A1 3 route de la révolle 93206 SAINT DENIS CEDEX  
Phone:+33-1-49-21-23-23 Fax:+33-1-49-21-23-24 E-mail:topcon@topcon.fr; www.topcon.fr

### TOPCON DEUTSCHLAND MEDICAL G.m.b.H.

Hanns-Martin-Schleyer Strasse 41; D-47877 Willich; GERMANY  
Phone:+49-(0)2154-8850 Fax:+49-(0)2154-885177 E-mail:med@topcon.de; www.topcon.de

### TOPCON SCANDINAVIA A.B.

Neongatan 2; P.O.Box 25; 43151 Mölndal, SWEDEN  
Phone:+46-(0)31-7109200 Fax:+46-(0)31-7109249 E-mail:medical@topcon.se; www.topcon.se

### TOPCON ESPAÑA S.A.

HEAD OFFICE  
Frederic Mompou 4 Esc. A Bajos 3, 08960 Sant Just Desvern Barcelona, SPAIN  
Phone:+34-93-4734057 Fax:+34-93-4733932 E-mail: medica@topcon.es; www.topcon.es

### TOPCON ( GREAT BRITAIN ) LTD.

Topcon House, Kennet Side, Bone Lane, Newbury, Berkshire RG14 5PX United Kingdom  
Phone:+44-(0)1635-551120 Fax:+44-(0)1635-551170 E-mail:medical@topcon.co.uk; www.topcon.co.uk

### TOPCON POLSKA Sp. z o.o.

ul. Warszawska 23; 42-470 Siewierz, POLAND  
Phone:+48-(0)32-6705045 Fax:+48-(0)32-6713405 www.topcon-polska.pl

### TOPCON SINGAPORE MEDICAL PTE. LTD.

1 JALAN KILANG TIMOR #09-01 PACIFIC TECH CENTRE SINGAPORE 159303  
Phone:+65-68720606 Fax:+65-67736150  
E-mail:medical\_sales@topcon.com.sg www.topcon.com.sg

### REPRESENTATIVE OFFICE IN INDONESIA

Level 38, Tower A, Kota Kasablanka unit GH-04, Jl. Casablanca, Kav 88, Jakarta, Indonesia 12870  
Phone:+62-21-2963-8004

### TOPCON INSTRUMENTS ( MALAYSIA ) SDN.BHD.

No. D1, (Ground Floor), Jalan Excella 2, Off Jalan Ampang Putra,  
Taman Ampang Hilir, 55100 Kuala Lumpur, MALAYSIA  
Phone:+60-(0)3-42709866 Fax:+60-(0)3-42709766

### TOPCON INSTRUMENTS ( THAILAND ) CO.,LTD.

77/162 Sinnsathorn Tower, 37th Floor, Krungthoburi Rd., Klongtong, Bangkok 10600, THAILAND  
Phone:+66(0)2-440-1152-7 Fax:+66(0)2-440-1158

### TOPCON CORPORATION BEIJING OFFICE

Block No.9, Kangding Street Beijing Economic-Technological Development Area,  
Beijing, 100176, CHINA  
Phone:+86-(0)10-6780-2799 Fax:+86-(0)10-6780-2790